

# **INSTRUCTION BOOKLET**

# VETERINARY METAL DETECTOR

## INSTRUCTION FOR USE:-

Metal detector is designed to predict very small metallic object. So at the time of operate the unit, the operator should not wear metallic items within a distance of approx. 100 cm to detector sensor rod. please remove finger rings, watches etc. also check no any metal shed or any other object in range of 100 cm.

- Take the detecting coil, meter unit cable and ear phone out of the bag.

- ◆ connect gray 5 pin cable between detecting coil and meter unit. and also connect ear - phone in meter unit.
- ◆ now switch the meter unit on. red on indication is displayed. now check battery charge level using battery testing switch.
- ◆ Sensitivity knob is 0 to 10. best setting is 3.
- ◆ now meter is ready to use and scale is display between 0 to 30.
- ◆ Start detecting tune sensitivity and listen the ear phone. When detect metal the volume is increase and meter scale is also increase.
- ◆ Charging batteries : charge battery with adaptor supplied with detector. when battery is discharge full the red on indication is dimming and off. and the working is not proper. charge battery for 7 to 8 hour for full charge.

VETERINARY METAL DETECTOR is a precision instruments especially for detecting covered magnetic and non magnetic metal parts. Magnetic metal will be found already in large distance according to their size, with non magnetic metal can be detected very close.